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rewetting drops
Preservative free
Multi Dose Units

- Biotrue® rewetting drops is a clear eye drop solution.
- Biotrue® rewetting drops is preservative free.
- Biotrue® rewetting drops can be used for 6 months once opened.
- Biotrue® rewetting drops contains the natural polymer, hyaluronic acid (as sodium hyaluronate), which is found in the structures of a healthy human eye and the natural tear film.
- Biotrue® rewetting drops is suitable for the wetting and rewetting of both soft and hard contact lenses during wearing and improves the comfort noticeably, all day long.
- Biotrue® rewetting drops is easy to use, well tolerated, and safe.

Please read these instructions for use leaflet carefully - they contain important information on how to use this product. If you have any questions, please ask your pharmacist, your doctor or contact us.

Composition:

The solution contains 0.24% hyaluronic acid (as sodium hyaluronate) as well as excipients including sterile water, sodium chloride, potassium chloride, disodium phosphate dodecahydrate and sodium dihydrogen phosphate-dihydrate. This product is preservative free.

Intended use of Biotrue® rewetting drops:

Eye lubricant which helps wetting and rewetting of both soft and hard contact lenses during wearing. Biotrue® rewetting drops is best suited for use to refresh and moisturise dry eyes caused by external environmental factors. Such factors are often influenced by lifestyle and surroundings including extensive computer use, indoor controlled-air environments, lack of blinking, cigarettes smoke, as well as windy outdoor environments.

Also, certain drugs (such as the contraceptive pill, antihistamines, betablockers, antispasmodics and diuretics), certain diseases or preservatives in some eye drops can play an important role in the development of ocular dryness.

Biotrue® rewetting drops hydrates the eye to relieve key dry eye symptoms like the feeling of dryness, irritation, or tired eyes or excessively tearing eyes.

Given the properties of this product, consumers with chronic dry eye can also benefit from its use.

How often should you use Biotrue® rewetting drops?

Instill one drop of Biotrue® rewetting drops into the eye as the need arises. As Biotrue® rewetting drops does not contain any preservative, you can use it every day and as often as you want.

Important information about using Biotrue® rewetting drops:

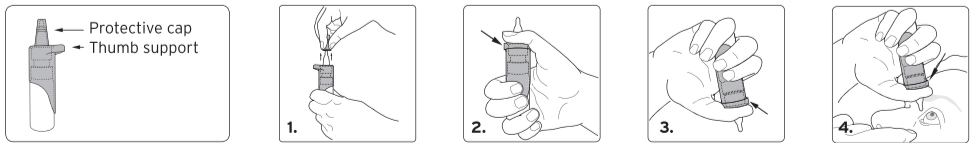
This product can cause hypersensitive reactions in very rare cases such as local allergic reactions. Treatment should discontinue in such cases. Do not use the product if you are hypersensitive to any of the ingredients.

Safety information for patients with severe damage of the cornea: In very rare cases deposits of calcium phosphate on the cornea were reported during treatment with phosphate-containing products for ophthalmic use.

In order to increase tolerability, Biotrue® rewetting drops does not contain a preservative. The pump mechanism allows you to use Biotrue® rewetting drops for 24 weeks (6 months) after opening. Please do not use Biotrue® rewetting drops after the expiry date.

Please, do not use if the bottle is damaged.

Please note that upon the first usage or after non-usage for a while, an initial pumping of the solution is necessary. By pumping the solution, the bottle should be kept with the dropper tip down.

1. Remove the protective cap from the tip of the bottle before each use. Make sure that the tip of the bottle does not touch your fingers.
2. Place your thumb on the ergonomic thumb support and two other fingers on the bottom of the bottle.
3. Return the bottle and hold it as shown. Before using the pump for the first time you must prime it. You can do this by pressing down on the thumb support approximately 7 times until the first drop appears.
4. To apply the drop, tilt your head back slightly and hold the bottle as preferred over your eyes. Gently pull your lower eye-lid down with a finger of your free hand, hold the bottle in a vertical position and gently squeeze the thumb support to apply one drop into the eye. Close your eyes immediately after the application and slowly roll them in order to optimally spread the drop on the surface of your eyes. You can use your other hand for additional support and accuracy if required.
5. Avoid contact of the tip of the bottle with your eyes or fingers to prevent any possible contamination of the solution.
6. Replace the protective cap on the tip of the bottle of the bottle immediately after use.

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Chem.-pharm Fabric GmbH
Importer: Einit Ltd.
Fabric GmbH Brunsbuttel Dam